

CODE: DUN001

Precision 88

P88 DIETARY ANTIGEN TEST

WHEN AND WHERE TO COLLECT

- The **blood sample** must be collected on **Monday or Tuesday MORNING** prior to 10am
 - We strongly encourage you to collect AND send your sample on a Monday. This provides a buffer of one day should there be any issue with courier pick up
- Sample must be sent by no later than Tuesday (or sample recollection may apply)
- The sample must be collected by Melbourne Pathology



MELBOURNE
PATHOLOGY

[CLICK HERE TO FIND A MELBOURNE PATHOLOGY COLLECTION CENTRE](#)

ON THE DAY

1. **Take the entire kit box with you to the collection centre**, including your frozen ice pack
2. Before you can return home with your sample, it must be spun in a centrifuge. The total process usually takes around one hour. Whilst waiting, arrange the courier
3. Take sample directly home/work and keep in fridge until courier collects
4. **Arrange the courier:**
 - a. Book a time for the courier to pickup sample by calling Team Global Express on **13 15 31**
 - b. Quote RN Labs' Customer/Billing Account Number **206342**
 - c. Tell the operator you have a **Pre-Paid Pick Up** & is a **Category-B Biological Substance**
 - d. Pick up address is preprinted on the label, suburb is Murarrie, QLD, 4172
 - e. The courier pick up window is usually before 5pm that day
 - f. **Someone must be present for collection to hand to courier**

4 Weeks before the test stop taking immunosuppressive drugs, this includes topical steroids

This is not a fasting test but you CAN be fasted if you are coordinating with another blood sample collection which requires fasting

Courier collection details

13 15 31

RN Labs account: 206342

Category B Biological Substance

Weight: 500grams

Dimensions: 20x20x7



READ THE INSTRUCTIONS INCLUDED IN THE TEST KIT BEFORE COMPLETING TEST AND BOOKING THE COURIER

Ensure sample is collected correctly.

The lab charges a \$50 retesting fee for incorrect samples